



CLEANSIFY NEO

Smart Wireless Ear Otoscope




Click  icon to search device, you may encounter prom like : WIFI can not connect to internet, keep the wifi or switch to another? (Prompt may be didderent depends on phones) Please select" keep the wifi " and give permission of using wifi

Designed with a built-in high-definition camera and soft silicone tip, Cleansify Neo enables precise visual inspection and cleaning of ears and other hard-to-see areas in real time using the EarVision app.

Accessories

1. Ear Otoscope 1 N
2. Charging Cable 1 N
3. User Manual 1 N
4. Silicone Ear Spoon Cover 5 N

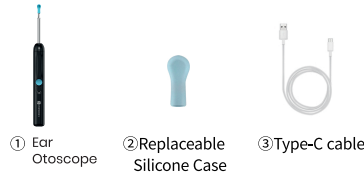

Note : Do not use charging equipment with output voltage exceeding 5V.

Precautions for use

1. After long time use, the lens will become dirty, please clean it regularly.
2. The temperature of the product will rise slightly (up to 35°C or less) It's normal.
3. After use, please use a cotton swab or soft cloth to clean the head of the ear spoon and the lens part. When cleaning the camera part, please be careful not to use excessive force.

4. Please observe your surroundings before use and do not use it with people running around to avoid impact.
5. If the ear canal has been completely blocked by cerumen, please be careful with ear picking and recommend to go to the hospital for professional treatment.
6. This product is not suitable for children under 3 years old.
7. Children are prohibited from using this product alone to avoid accidental injury.
8. Please do not use the camera in liquid to avoid damage caused by water immersion.
9. Avoid direct exposure of the product to sunlight, especially silicon to avoid softening.
10. Please do not knock or drop the product, if the product is damaged, please stop using it.
11. Please charge the product regularly if it is not used for a long time.
12. Please do not disassemble, modify or repair this product by yourself.

How to Download

Download:  EarVision

1. Apple download method: 



Please search for in the Apple AppStore to download.

2. Android download method:




Please find it on Google Play Search for EarVision to download.

Switching on the Power


- Press & hold the power button for two seconds to turn on the indicator turns blue.
- When the indicator is blinking, it means the machine is not connected.
- When connected LED becomes stable.
- When no devices connected for more than 3mins, Cleansify Neo turns off.

Usage


Directions for use

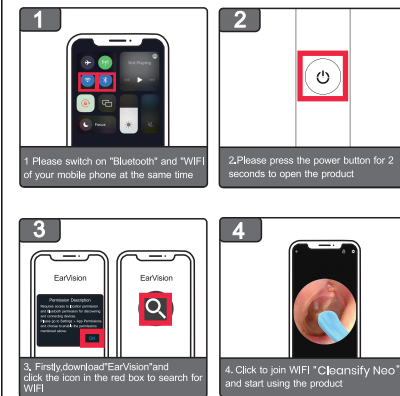

- ① When you start the application for the first time, you will be prompted for "Privacy Policy". Please select "Agree"

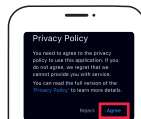

- ② When opening App, Phone Prompts : "EarVision" would like to find and connect to devices on your local network please select "Allow"

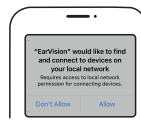

- ③ When opening App, Phone Prompts : "EarVision" Would Like to Use Bluetooth please select "OK"




- ④ When opening App, Phone Prompts : Allow "EarVision" to use wireless data? Please select "WLAN Only"



- ⑤ When opening App, Phone Prompts : "EarVision" Wants to Join WLAN Network "softish-4c5061"? please select "Join"




- ⑥ You will have prompt, whether to allow EarVision to get location information. Please select "Precise Location" and "Allow only during use"

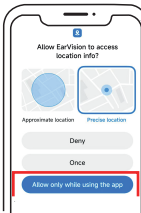

※ Without revealing your location information, Only for connecting to the product WIFI

※ If wifi connection can not established, restart the machine and run app again.

App Screen Description

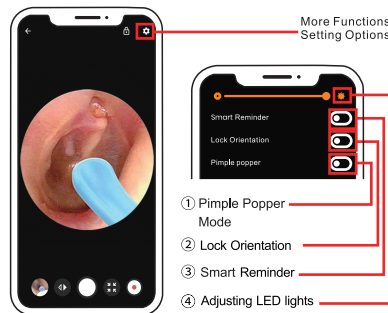

1. Smart Reminder : Acts as a protection, to avoid contact with eardrums
2. Lock Orientation : Locks the silicone in the camera screen, helps for better viewing before cleaning
3. Pimple Popper Mode : Front camera turns ON and displays for easy usage.
4. To adjust LED brightness.

Icons Description



- Record Button
- Image Archive
- Zoom Screen
- Settings: Adjust Light, Lock Direction, Squeeze Pimple
- FAQ
- Photo Album
- Privacy Policy
- Search WIFI

Charging

- Use the provided Type-C to USB cable for charging. When charging starts, the Indicator will light up red.
- When fully charged, the indicator will turn off.

Product Parameter

Input voltage/ Current	DC 5V/200m A
Charging time	45 mins
Usage time	60 mins
Battery mAh	130mAh
Camera Pixel	Ultra HD Camera



E-Waste Management
E: info@3recycler.com

For E-Waste & ROHS compliance
refer to www.portronics.com/ewaste

www.portronics.com
For Support

Visit : www.portronics.com
Email : help@portronics.com

Contact No. : +91 9555245245

(Working Hours. : Monday - Saturday 10:00am - 6:00pm)